
Instructions for Use





---









REBOUND<sup>®</sup> POST-OP KNEE

Cool Foam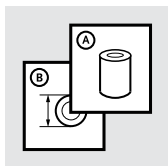

		3
EN	Instructions for Use	5
DE	Gebrauchsanweisung	7
FR	Notice d'utilisation	9
ES	Instrucciones para el uso	11
IT	Istruzioni per l'uso	13
NO	Bruksanvisning	15
DA	Brugsanvisning	17
SV	Bruksanvisning	19
FI	Käyttöohjeet	21
EL	Οδηγίες Χρήσης	23
NL	Gebruiksaanwijzing	25
PT	Instruções de Utilização	27
PL	Instrukcja użytkowania	29
CS	Návod k použití	31
TR	Kullanım Talimatları	33
RU	Инструкция по использованию	35
JA	取扱説明書	37
ZH	中文说明书	39
KO	사용 설명서	41
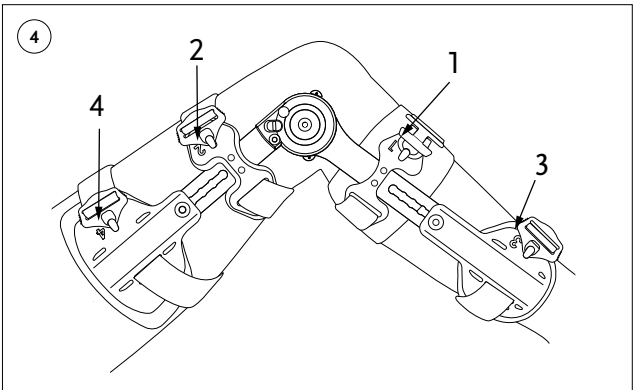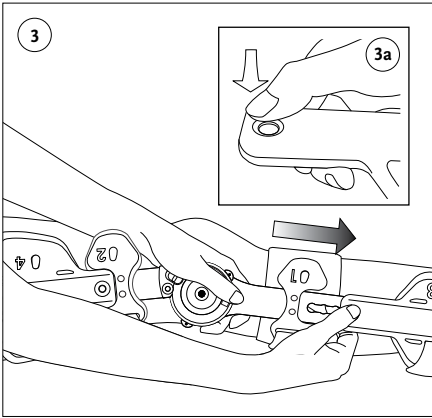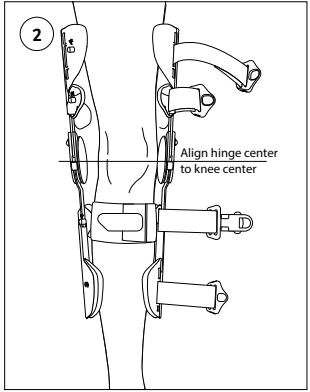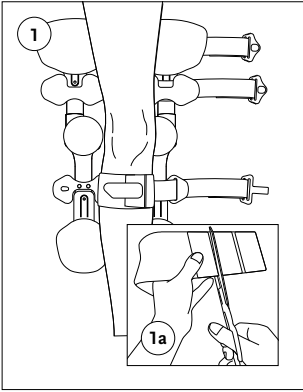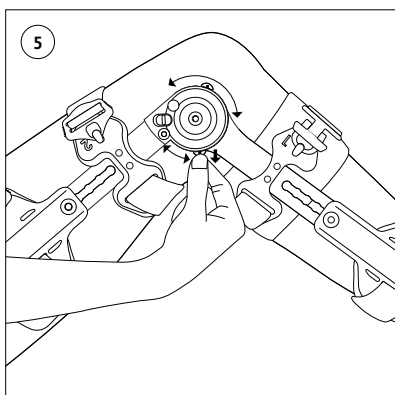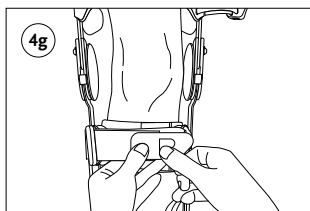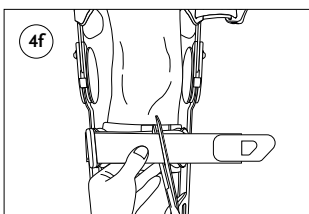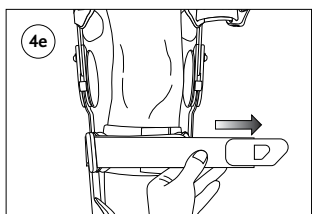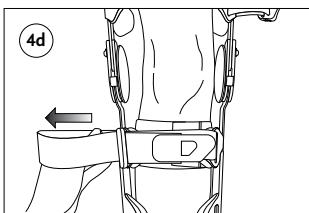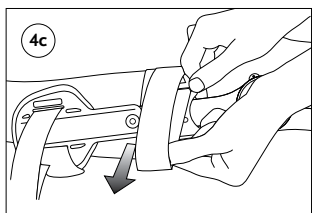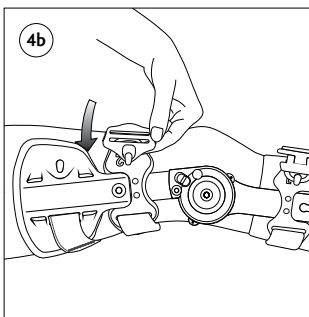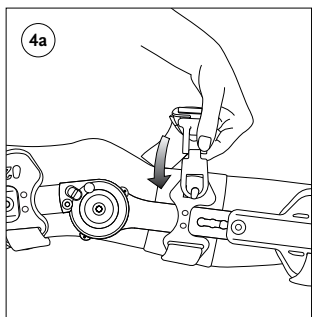
# ENGLISH

---

## INDICATIONS FOR USE

The Rebound® Post-Op Knee brace is designed to limit flexion and extension and may be beneficial in the following situations:

- ACL, PCL, MCL, LCL repairs/injuries
- Meniscal repairs
- Tibial plateau fractures
- Patellar tendon repairs
- Osteochondral repairs
- Condylar fractures
- Sprains/strains of the knee
- HTOs (High Tibial Osteotomy)

## INTENDED USE

Controlled range-of-motion for post-traumatic, post-surgical repair of the knee.

## CAUTION

1. This product has been designed and tested for single patient use, and is not recommended for multiple patient use.
2. The patient must immediately contact their practitioner and discontinue use if:
  - any problems occur with the use of this product;
  - the patient experiences increased pain, swelling, sensation changes, skin irritation, or other adverse reaction while using this product.
3. The patient should be advised to consult their practitioner before making any adjustments to the brace.
4. This product is intended to be prescribed and fitted by a licensed practitioner.
5. Proper rehabilitation and activity modification are also essential parts of a safe treatment program.

**Note:** The Rebound Post-Op knee brace is not made with natural rubber latex.

## FITTING INSTRUCTIONS

1. Apply anti-migration strap (**Figure 1**). Cut to size along groove if necessary (**Figure 1a**).
2. Position brace paddles on leg by aligning hinge center to knee center (**Figure 2**).
3. To adjust length of hinge arms: (**Figure 3**).
  - Push down on length adjustment button to unlock paddle (**Figure 3a**).
  - Slide paddle to desired length.
  - Repeat adjustment for all paddles. Ensure all length adjustment buttons are in locked position.

**Important:** Practitioner should adjust the brace length to provide optimal knee stabilization. Upper hinge arms should extend no higher than 5 cm from groin level. Lower hinge arms should extend no lower than calf/Achilles tendon junction.
4. Fastening Straps (**Figure 4**):
  - Secure buckles (**Figure 4a, 4b**)

- Tighten strap (**Figure 4c to 4e**)
  - Straps may be trimmed if necessary (**Figure 4f to 4g**)
5. To adjust range-of-motion, pull the flexion or extension button and move to desired angle printed on hinge and release (**Figure 5**).
  6. Place ROM stop locks over flexion and extension stops to limit range of motion adjustment.

#### **DROP-LOCK INSTRUCTIONS**

- Determine the desired lock angle
- Slide drop-lock button toward center of hinge to lock the hinge angle

#### **WASHING INSTRUCTIONS**

- Do not wash in washing machine or place in dryer
- Use mild detergent and cold water (30°C/86°F)
- Remove pads and wash separately
- Do not immerse hinge & uprights in water
- Do not use bleach or strong detergents

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