








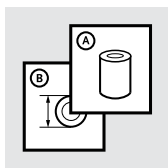
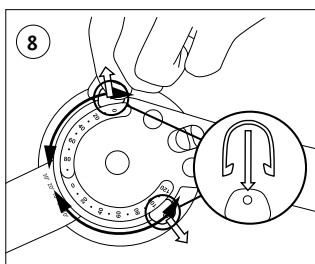
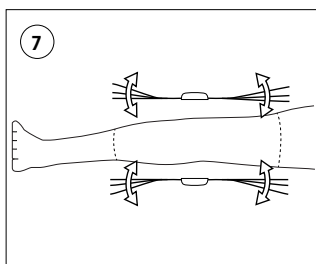
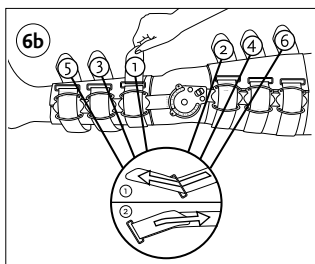
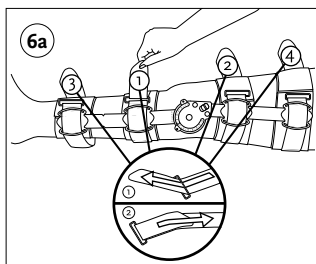
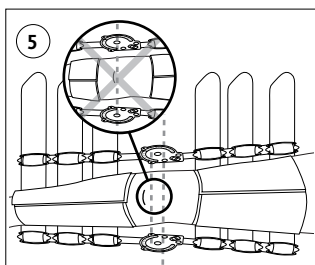
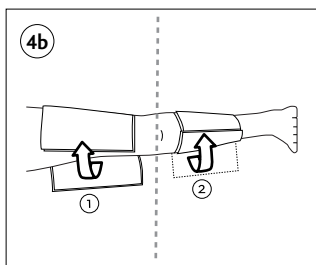
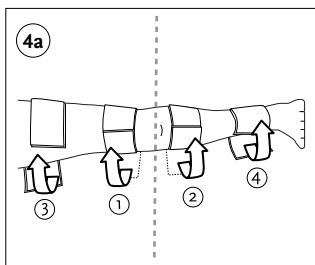
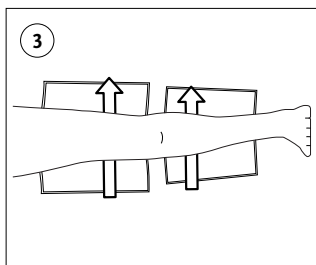
Instructions for Use

FORMFIT[®] POST-OP KNEE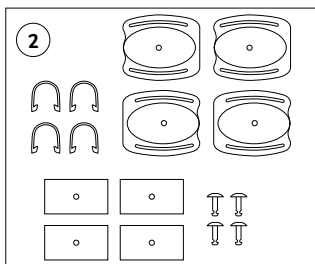

		3
EN	Instructions for Use	5
DE	Gebrauchsanweisung	9
FR	Notice d'utilisation	13
ES	Instrucciones para el uso	17
IT	Istruzioni per l'uso	21
NO	Bruksanvisning	26
DA	Brugsanvisning	30
SV	Bruksanvisning	34
EL	Οδηγίες Χρήσης	38
FI	Käyttöohjeet	42
NL	Gebruiksaanwijzing	46
PT	Instruções de Utilização	50
PL	Instrukcja użytkownika	54
CS	Návod k použití	58
TR	Kullanım Talimatları	62
RU	Инструкция по использованию	66
JA	取扱説明書	70
ZH	中文说明书	74
KO	사용 설명서	79
SK	Návod na použitie	83
ET	Kasutusjuhend	87
RO	Indicații pentru Utilizare	91
AR	تعليمات الاستخدام	95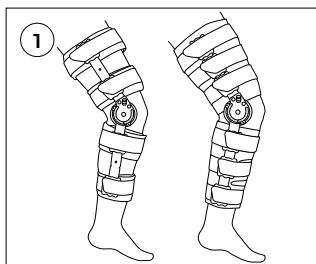

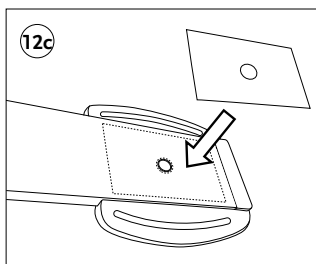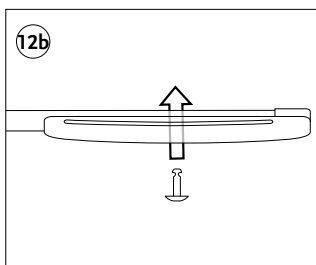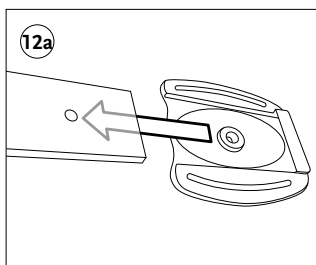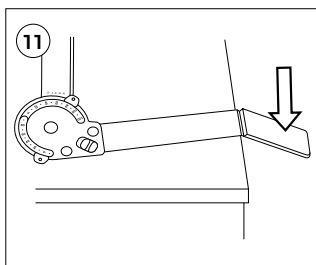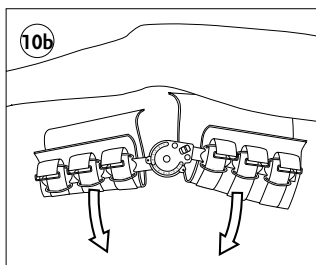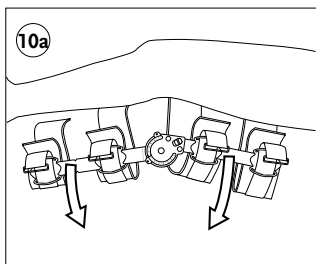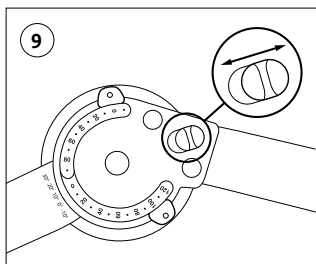ENGLISH

PRODUCT DESCRIPTION

The Formfit® Post-Op Knee 24" (61 cm) and 28" (71 cm) braces control the range of motion (ROM) and immobilize the knee after surgery or injury (Figure 1). The two braces are available in the sizes regular and XL.

Cool and Full Foam

They are also available as cool foam and full foam versions. The cool foam version has four foam pads. The full foam version has two foam pads that are wider.

Shortening

If necessary, the two braces can be shortened at the distal and proximal end. A shortening kit is included for the cool foam versions of the brace (Figure 2). A shortening kit is not necessary for a full foam brace.

Straps

You can order strap length reduction kits for patients with small thigh circumferences.

Replacement softgoods and accessories can also be ordered separately.

Sizes

Type	Thigh Circumference	Cool Foam	Full Foam
Formfit® Post-Op Knee 24" Reg	Up to 27" / 69 cm	FPOK403100	FPOK403200
Formfit® Post-Op Knee 24" XL	27–35" / 69–89 cm	FPOK403102	FPOK403202
Formfit® Post-Op Knee 28" Reg	Up to 27" / 69 cm	FPOK403101	FPOK403201
Formfit® Post-Op Knee 28" XL	27–35" / 69–89 cm	FPOK403103	FPOK403203

Replacement Softgoods

Type	Cool Foam	Full Foam
Formfit® Post-Op Knee 24" Reg	403175	403275
Formfit® Post-Op Knee 24" XL	403176	403277
Formfit® Post-Op Knee 28" Reg	403175	403276
Formfit® Post-Op Knee 28" XL	403176	403278

Accessories

Type	Order Number
ROM stop locks (set of 20 lock clips)	B-8380005
Strap Reduction Kit Cool	403199
Strap Reduction Kit Full Foam	403299

INDICATIONS FOR USE

Post-operative/post-injury ROM control or immobilization.

Ligament and fracture-related surgeries and when controlled range of motion is necessary.

CONTRAINDICATIONS

None known.

INTENDED USE

The Formfit® Post-Op Knee is intended to be used for knee immobilization and range of motion restriction.

GENERAL SAFETY INSTRUCTIONS

WARNING: Do not use for multiple patients. Only use the device for a single patient.

CAUTION: If you experience increased pain, swelling, sensation changes or any adverse reactions while using this product, immediately consult your medical professional.

USA Only: Federal law restricts this device to sale by or on the order of a licensed health care practitioner.

USING THE BRACE

Size Selection

Measure the thigh circumference 6" / 15 cm above the mid patella.

Select a size that agrees with the measurement. Refer to the "Sizes" chart in the section "Product Description" to find the correct size.

DONNING THE BRACE

WARNING: Make sure to keep the leg straight when you slide the foam pads and straps under it.

1. Unfasten the leg straps.
2. Fully pull the open ends of the straps through the double D rings to remove them from the left strut.
3. Remove the foam pads from the two struts.
4. Fully open the foam pads.
5. Carefully slide the foam pads under the thigh and calf (**Figure 3**).
6. Fasten the foam pads around the leg in the correct sequence.
Cool foam: figure 4a. Full foam: figure 4b.
7. Put the struts on each side of the leg.
8. Align the top of the hinge pivot with the top of the patella (**Figure 5**).
9. Align the foam pads with the hook and loop fasteners on the inner surface of the struts.
10. Attach the struts to the foam pads on each side of the leg.
11. Carefully slide the straps under the foam wraps.
12. Put the straps back through the double D rings.
13. Pull and fasten the straps around the foam pads in the correct sequence.
Cool foam: figure 6a. Full foam: figure 6b.

CAUTION: Do not tighten the straps too much. If the straps are too tight, the blood circulation can be cut off.

14. If necessary, use bending tools to adapt the brace to the patient's leg shape (**Figure 7**).

Setting Range Of Motion

Extension can be limited at: -10° , 0° , 20° , 30° , 40° , 50° , 60° , 70° , 80° , 90° .

Flexion can be limited at: 10° , 20° , 30° , 40° , 50° , 60° , 70° , 80° , 90° , 100° , 110° , 120° .

Adjust ROM as follows (Figure 8):

Extension: Pull the black extension button up and set the angle.

Flexion: Pull the white flexion button up and set the angle.

Release a button to let it click into place. Put the stop locks onto the buttons to secure them.

Immobilizing the Knee

Use the drop lock to fully immobilize the knee. It can be set at -10° , 0° , 10° , 20° , 30° of flexion. An arrow shows the angle. The angle is shown by the scale on the strut, below the hinge.

Adjust immobilization as follows (Figure 9):

1. Push the blue drop-lock button up to unlock the hinge.
2. Move the brace until the arrow and the correct angle agree.
3. Push the blue drop-lock button down to lock the hinge angle.

Doffing the Brace

1. Unfasten the straps and the foam pads.
2. Remove the brace (**Figure 10a**, **10b**). You can put it back on again fully assembled.

SHORTENING THE BRACE

If necessary, the 24" and 28" braces can be shortened to 16".

WARNING: Only shorten the brace if ordered by the treating doctor. Shortening the brace reduces the function of the brace.

To shorten the two braces, break off the distal and proximal ends of the struts at the seams. Use the shortening kit for a cool foam brace. If you shorten a full foam brace, you only need to trim the foam pads.

Shorten the struts as follows:

1. Remove the brace from the patient.
2. Fully remove the straps and foam pads from the struts.
3. Put a strut on a table.
4. Align a seam with the edge of the table.
5. Hold the strut tight and break off at the seam (**Figure 11**).
6. Repeat for the other end.

Full Foam

1. Make sure the edges are smooth. If necessary, use a file to round the edges.
2. If necessary, use scissors to trim the foam pads.
3. Assemble the brace again (refer to "*Donning the Brace*" above).

Cool Foam

Use the shortening kit as follows:

1. Put the double D-rings from the kit onto each shortened end of the struts. Make sure the double D-rings are on the outer side (**Figure 12a**).
2. Put the plastic rivet through the hole and the double D-ring. The top of the rivet must point to the outer side (**Figure 12b**).
3. Attach the hook and loop fastener over the rivet (**Figure 12c**).
4. Assemble the brace again (refer to “*Donning the Brace*” above).

PRODUCT CARE

The foam pads can be hand washed with mild soap. Leave to dry. Do not tumble dry.

PRODUCT MATERIAL LIST

Polyamide, polyamide with glass fiber, polyurethane, steel, stainless steel, aluminum, zinc.

DISPOSAL

All components of the product and packaging should be disposed of in accordance with respective national environmental regulations. Users should contact their local governmental office for information on how these items can be disposed of in an environmentally sound manner.

WARRANTY

Plastic parts and hinges 3 months warranty. Foam pads and leg straps 30 days. 6 months free from defects.

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